

This is the free software link to our website that explains how to download to your computer. This Review Express is just for downloading events....

http://www.medical.philips.com/us/products/resuscitation/products/event_review/free_reviewexpress.html

PHILIPS EVENT DATA: DOWNLOAD:

With Review Express Software, you can download ECG data from your defibrillator's* data card or infrared (IR) port, view it on your PC screen, print it, and erase your data card or defibrillator's internal memory. If you need additional capabilities, please consider purchasing HeartStart Event Review.

System Requirements for Review Express

Computer/ Processor - Pentium III equivalent or higher

Display - Super VGA 800x600 16-bit color minimum. 1024x768 screen resolution with 16-bit color recommended

Memory - 128 MB recommended (64 MB minimum)

Operating System - Windows 2000 Professional or Windows XP Professional. For Windows XP and Windows 2000, Review Express 3.1 must be installed by user with administrator rights.

Web Connection - A modem or broadband Internet connection is required for download. File is approximately 7.1 megabytes in size - download time will vary depending on your connection speed.

[CLICK HERE TO INSTALL](#)

Complete Survey, Click Submit
Click RUN Exe.
Install