



ZOLL AED PLUS: AED PRO:

With RescueNet™ Code Review, you only need the AED and a PC with an IrDA port (*ZOLL also sells adapters if you don't currently have an IrDA port*).

RescueNet Code Review electronically captures and organizes resuscitation data collected from ZOLL defibrillators. Code Review provides ECG waveform and CPR rate and depth of compression review to improve training and processes

Go to: www.zolldata.com/web/download.htm.

Click on “English” Version, Free Download

This may take a few seconds to download. Click “Install Shield Wizard”

1. Position the AED Plus or AED Pro so that its infrared port lines up with the RescueNet Code Review workstation's infrared port or reader.
2. In Code Review, click the Upload Case from Infrared button.
3. On the defibrillator, press and hold the On button.
4. Keep holding the On button until you hear the defibrillator say, "Non-rescue mode. Communications established."
5. In Code Review, a dialog box displays the transfer progress.